

# OrthoMed Hot or Cold Versatile Insulated Gel-Pack

The OrthoMed hot or cold versatile insulated gel-pack. Can be heated in a microwave for warm therapy or placed in a freezer for cold therapy. Contents are non-toxic and nonirritating. Soft-freeze packs stay flexible when frozen to easily mold to the injured area.

- Available in three sizes
- Remains flexible
- Non-toxic
- Reusable



*Insulated side should be positioned away from the body making the pack more comfortable to hold. Insulation extends the therapeutic time duration of the pack by approximately 50%.*

No.	Description	Size
OM 300225	Small 325g (non-insulated)	4.5" x 10"
OM 300228	Small 325g	6" x 9"
OM 300230	Large 800g	8" x 15"

## For use as a Hot Pack

**To boil:** Bring 4 quarts of water to a rolling boil. Remove water from heat and immerse room temperature gel-pack. Remove pack from water, with tongs, after 40 seconds. Wrap with towel and apply to desired area.

**To microwave:** Place room temperature gel-pack in microwave. Heat at full power (750 watts) for no more than 55 seconds. Remove pack from microwave. Knead pack to distribute heat evenly. For additional heat, reheat pack in microwave at 10 second intervals. Repeat as necessary to reach desired temperature. Do not exceed 65 seconds total heating time. Wrap with towel and apply to desired area. Microwave ovens vary in wattage output. Some may heat at a faster or slower rates.

## For use as a Cold Pack

Place gel-pack flat in freezer for a minimum of two hours. Remove gel-pack from freezer. Wrap with towel and apply to desired area.

*Instructions in English and Spanish.*

### **We are OrthoMed Inc.**

OrthoMed, Inc. has been providing health care professionals quality products and services since 1970. Founded by Ronald E. and Martha Dyches who built a successful business on the values of bringing innovative products to the market and providing the highest level of service possible.

We continue with that commitment today, providing health care professionals with the highest quality products and services at the most competitive prices. We pride ourselves on being the Surgical Instrument Specialists and being a complete source for all your instrument needs. We cross reference most instrument companies and can often source the unique and hard to find instruments not found in this catalog. We are happy to do the research if you provide a cross reference number or a picture.

### **OrthoMed, Inc.**

#### **The Surgical Instrument Specialists**

We are proud to introduce this catalog with more than 3,300 products featuring one of the most comprehensive instrument lines in the industry. At OrthoMed, Inc. we have a passion for surgical instruments and truly want to be known as the Surgical Instrument Specialists. At OrthoMed, Inc. we will continue to provide the innovative instruments that surgeons demand for the ever evolving new surgical techniques. We also have a large number of spinal instruments for this growing specialty.

In addition to our premium Operating Room instruments we provide middle and economy grade instruments for Central Supply, the emergency room, and the floors. These instruments also must pass strident manufacturing processes and are ISO 9001 certified in conjunction with FDA and CE standards. Many instruments used in these areas may not need to be premium grade when a lower cost instrument will suffice.

Our goal at OrthoMed, Inc. is to provide all your surgical instrument needs and continue to build on the trust and confidence that we have earned over the last 36 years.

### **Ordering**

Orders may be placed by contacting our customer service department at **503 234-9691** or toll free **800 547-5571**. Orders can also be faxed to **503 234-8629** or emailed to **custserv@orthomedinc.com**. OrthoMed, Inc. also has authorized sales representatives and distributors. Please call customer service for the sales representative or distributor serving your area.

### **Return Policy**

OrthoMed, Inc. requires a return goods authorization on all returns. Most returns will be authorized unless they are over 30 days from date of invoice, have been used, processed or sterilized, or are special or custom orders. A restocking fee may be charged for items with missing packaging or labeling. To receive authorization for the return of unused merchandise contact your OrthoMed, Inc. sales representative or call customer service at 800 547-5571.

### **Guarantee**

OrthoMed, Inc. instruments are guaranteed for life against manufacturing defects of material or workmanship. This guarantee is void if instruments are not maintained or repaired properly, or if they are not used for their intended surgical purpose. Tungsten carbide inserts on scissors, needle holders, and wire cutters are guaranteed for three years. Replacement parts, such as springs, are guaranteed for one year.

OrthoMed, Inc.  
3208 SE 13th Avenue  
Portland, Oregon 97202

503.234.9691  
800.547.5571  
[www.orthomedinc.com](http://www.orthomedinc.com)  
[custserv@orthomedinc.com](mailto:custserv@orthomedinc.com)